

Collection Device

This list may not contain all physician requests and acceptable substitute collection supplies may be provided when devices shown are not available. Please refer to the online Aspirus Reference Laboratory Test Directory (testmenu.com/Aspirus) for the most comprehensive and up to date specimen collection information.

Test

Culture, Aerobic
 Culture, Anaerobic with Aerobe
 Culture, Yeast
 Culture, Eye
 Culture, Gonococcus
 Culture, Ear
 Staph aureus Culture
 Culture, Vancomycin-Resistant Enterococcus
 Culture, Methicillin Resistant Staphylococcus aureus
 Group B Strep NAT, Amplified
 Culture, Fungus
 Fungal Smear
 Staph aureus & MRSA Detection by NAT
 Wet Prep, Genital

Collection Device - ESwab Regular



Test

Bordetella pertussis/parapertussis, NAT Amplified

Collection Device - SWAB-PAK



Test

Culture, Nasopharyngeal
 Pediatric patient cultures

Collection Device - ESwab Flex Mini



Test

Vaginal Panel NAT, Amplified

Collection Device - BD Molecular Swab Kit



Test

Rapid Strep Group A with Reflex to Culture


Collection Device - ESwab Regular



Test

Viral Culture
 Culture, Herpes Simplex Virus
 Culture, Viral, Non-Respiratory
 Culture, Chlamydia trachomatis
 Culture, Mycoplasma & Ureaplasma

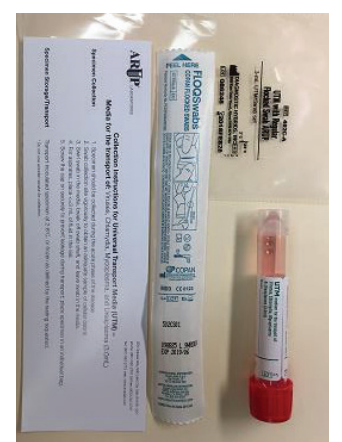



Collection Device - ARUP Transport Media & Swab Kit



Test

Chlamydia trachomatis NAT, Amplified
 Neisseria gonorrhoeae NAT, Amplified
 Trichomonas vaginalis NAT, Amplified
 Eye/Oral/Rectal: Miscellaneous Lab Test, Modifier 0060241

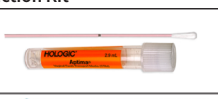Collection Device - Aptima Collection Kits

Source	Collection Kit	
<ul style="list-style-type: none"> Provider or Patient collected vaginal swab Provider collected rectal swab (not for T. vag NAT) Provider collected pharyngeal swab (not for T. vag NAT) 	Aptima Multitest Swab Collection Kit	
<ul style="list-style-type: none"> Provider collected Male urethral swab (not for T. vag NAT) Provider collected endocervical swab 	Aptima Unisex Collection Kit (Use the blue-shaft swab for sample collection.)	
Male or Female Urine	Aptima Urine Collection Kit <ul style="list-style-type: none"> Transfer approximately 2ml of first-catch urine into Aptima urine collection tube. Urine volume must be between the black indicators on the tube. Minimum volume = 2 ml 	
Liquid Cytology	Aptima Specimen Transfer Kit <ul style="list-style-type: none"> 1 ml of a well-mixed ThinPrep or SurePath liquid cytology specimen transferred to an Aptima Specimen Transfer Tube. 	

Test

Influenza A/B, NAT, Amplified
 Influenza A & B Rapid Antigen
 Influenza A/B, RSV, NAT, Amplified
 SARS-CoV2, NAT, Amplified
 SARS-CoV-2 & Influenza A/B NAT, Amplified
 SARS-CoV-2, Influenza A/B, & RSV NAT, Amplified
 RSVE
 RSV, NAT, Amplified

Collection Device - Nasal or NP Swab UTM (Universal Transport Media)

